

N95 User Seal Check - Every Time You Wear It



Cannot obtain proper seal, DO NOT enter care area.

Cup Style Respirator

- After applying mask, place both hands completely over the mask being careful not to disturb the position.
- Inhale and exhale forcefully. If air leaks round the nose, adjust the nosepiece.
- If air leaks around the edges, adjust the side panel and straps along the sides of your head.
- Repeat seal test after adjusting the mask.



[Click Here for Video](#)

Duckbill Style Respirator

- After applying mask, forcefully inhale and exhale several times.
- Mask should collapse slightly when inhaling and expand when exhaling.
- No air should be felt leaking between your face and the mask.
- Adjust mask until leakage is corrected, and repeat seal test.

